




Sending Images and Scans from Planmeca Romexis[®] to Henry Schein DDX

Creating a New Case

Creating a New Case

Select the case from the case list in Romexis. Click **DDX Export**.

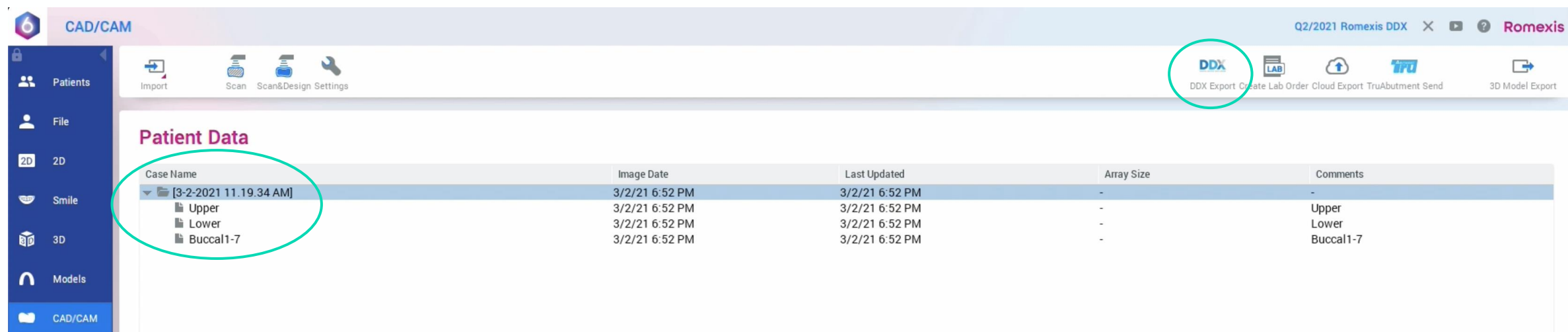


The screenshot displays the Romexis CAD/CAM software interface. The top navigation bar includes the Romexis logo and the text "Q2/2021 Romexis DDX". The left sidebar contains navigation options: Patients, File, 2D, Smile, 3D, Models, and CAD/CAM. The main workspace is titled "Patient Data" and contains a table with the following data:

Case Name	Image Date	Last Updated	Array Size	Comments
[3-2-2021 11.19.34 AM]	3/2/21 6:52 PM	3/2/21 6:52 PM	-	-
Upper	3/2/21 6:52 PM	3/2/21 6:52 PM	-	Upper
Lower	3/2/21 6:52 PM	3/2/21 6:52 PM	-	Lower
Buccal1-7	3/2/21 6:52 PM	3/2/21 6:52 PM	-	Buccal1-7

The toolbar at the top right of the workspace contains several icons. The "DDX" icon, which stands for "DDX Export", is circled in green. Other icons include "LAB", "Cloud Export", "TruAbutment Send", and "3D Model Export".



Creating a New Case

The DDX dialog box will open. Select the file format for the scans and click **Create New Case**.

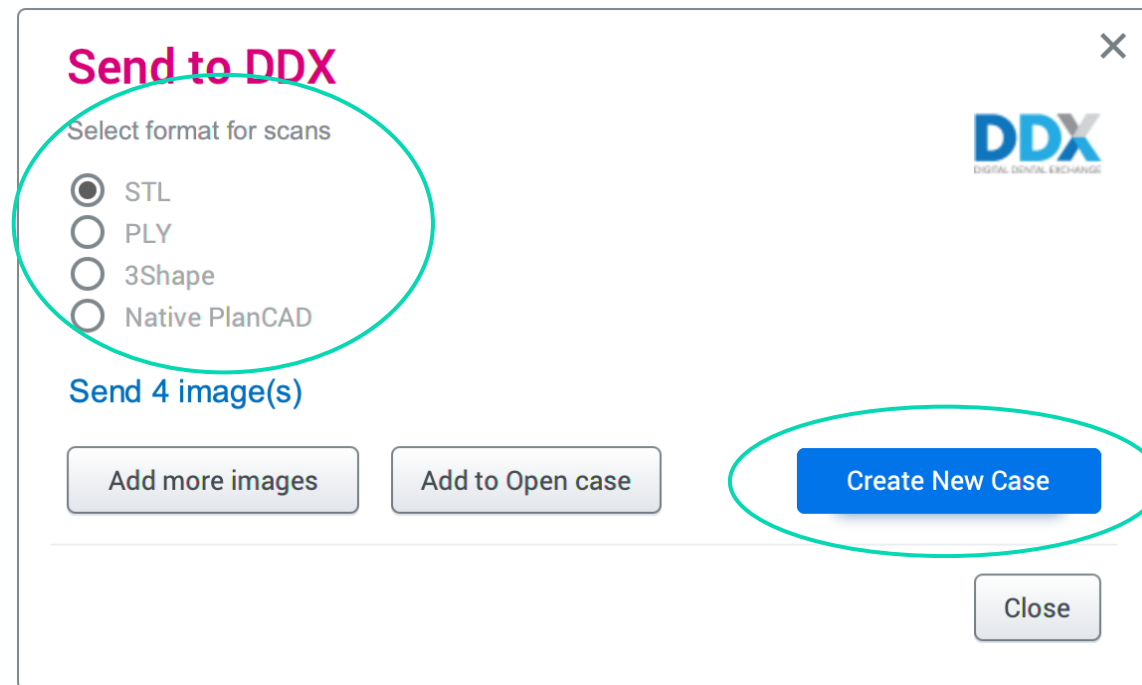

Creating a New Case

If you would like to add additional images before sending the case, select **Add More Images**. Select the images you would like to add and click **Update Case**.

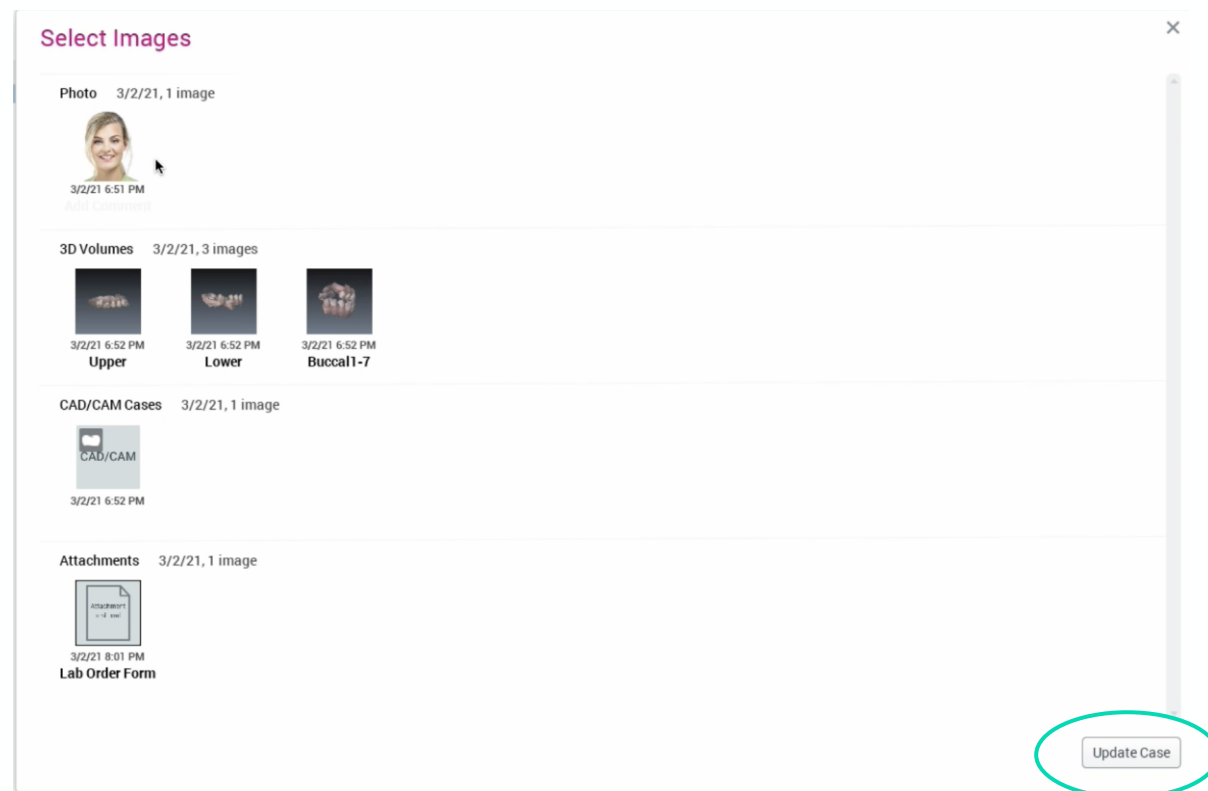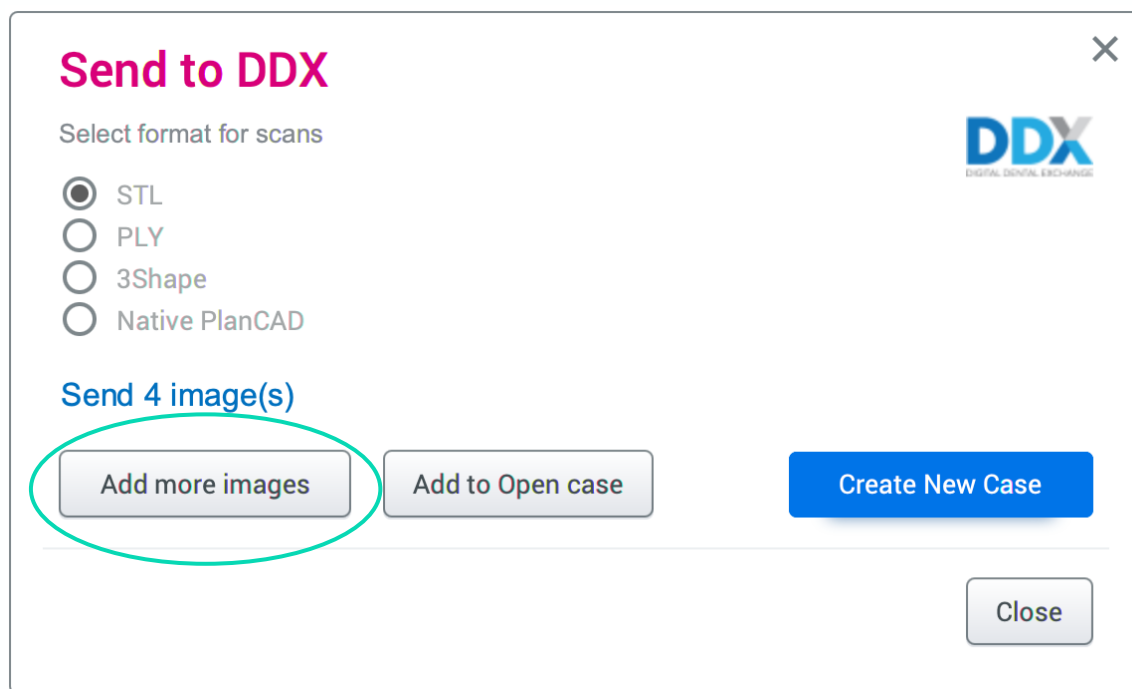

Creating a New Case

The DDX portal will immediately launch automatically.
Log-in or Sign-up for a new account.

DDX DIGITAL DENTAL EXCHANGE

Sign Up Login

Login

Login as

Email *

Password *

Security

- This is a public or shared computer
- This is a private computer

[→ Login](#)

[Lost your password?](#)

Don't have a DDX account? [Sign Up](#)

Creating a New Case

Select the desired lab from the list of **My Labs** or **Find A Lab**.
Click **Use Lab**.

The screenshot displays the DDX Digital Dental Exchange user interface. At the top left is the DDX logo with the tagline 'DIGITAL DENTAL EXCHANGE'. On the top right, there are links for 'Invite a Colleague', 'Help', and a user profile for 'Dr Jane Doe @ Jane Doe Dental'. Below the header is a blue navigation bar containing 'Portal', 'Go To My Partners', 'New Task', and a search field. The main content area is titled 'Schedule a New Case' and includes a 'Select a Lab' section. This section is divided into two panels: 'My Labs' and 'Find A Lab'. The 'My Labs' panel shows a list with 'Reveal Aligner Lab' selected. The 'Find A Lab' panel has search filters for 'Finland' and 'Lab Name'. Below these panels is a 'Lab Details' section for 'reveal CLEAR ALIGNERS', listing 'Reveal Aligner Lab', 'Eagan, MN', and '1-888-888-8888'. A 'Use Lab' button with a checkmark is located at the bottom right of the lab details section. Red circles highlight the 'Schedule a New Case' title and the 'Use Lab' button.

Creating a New Case

Enter any information needed from DDX (Provider information, Procedure, Teeth) and Click **Submit Case**.

reveal
CLEAR ALIGNERS

Invite a Colleague Help Dr. Jane Doe - @ Jane Doe Dental

Portal Reveal Aligner Lab Cases Finances

New Task

Search

Schedule a New Case

Patient Information

Provider *	Gender
Dr Jane Doe	Male
First Name	Chart Number *
Javier	7123
Last Name *	Birth date
Demo	

Work Requested

Add Procedure

Procedures

Case Files Upload

No files uploaded

Notes

Subject

Apply To...

0 / 1000

Send Date

2021-03-04

Try-in Emergency Pickup

Tags

Submit Case

Save Draft

Creating a New Case

When the images have successfully uploaded to the DDX portal, you will receive a pop-up notification.



Creating a New Case

You will now be able to view the case on the DDX portal.

The screenshot displays the DDX reveal portal interface. At the top left is the logo for DDX reveal CLEAR ALIGNERS. The top right navigation bar includes links for 'Invite a Colleague', 'Help', and the user profile 'Dr Jane Doe - @ Jane Doe Dental'. Below this is a blue navigation bar with tabs for 'Portal', 'Reveal Aligner Lab', 'Cases', and 'Finances'. A 'New Task' button and a search bar are also present. The main content area is titled 'Cases' and features a sidebar with 'New Case', 'By Month', and 'Open Cases' options. The central focus is 'Case #93', which has a 'Print' and 'Cancel' button. The case details are as follows:

- Status:** DDX Case Waiting to Arrive
- Return Date:** Apr 12, 2021
- Send Date:** Mar 16, 2021
- Provider:** Dr Jane Doe
- Case Monitoring:** Not Following (checked)
- Patient:** Javier Demo
- Patient Appointment:** Not Entered
- Arrived by You:** Not Arrived (checked)
- Tags:** None

On the right, the 'Patient Appointment' section includes fields for 'Date' and 'Time', a 'Set' button, and a list of actions: 'Reschedule Send Date', 'Add Procedure to Case', and 'Attach Files'. Below the case details is a 'Details' section with 'Notes (0)' and a 'Work Requested' section. At the bottom, there is a table for 'Units' and 'Description' with one entry: '1 Reveal Clear Aligners'. A 'Case Files' section with an 'Upload' button and a note '12 Specified Files Recommended' is also visible.

Note: Cases are saved in numerical order, not by patient name or ID.
To view the images, you may need to refresh the DDX page.



Sending Images and Scans from Planmeca Romexis[®] to Henry Schein DDX

Adding to an Existing Case

Adding to an Existing Case


Select the case from the case list in Romexis. Click **DDX Export**.

The screenshot displays the Romexis CAD/CAM software interface. The top navigation bar includes the Romexis logo, the text 'Q2/2021 Romexis DDX', and a 'Romexis' label. Below this is a toolbar with icons for 'Import', 'Scan', 'Scan&Design Settings', and 'Settings'. On the right side of the toolbar, the 'DDX' button is circled in green, with a tooltip that reads 'DDX Export Create Lab Order Cloud Export TruAbutment Send'. Other buttons in the tooltip include 'LAB', 'Cloud Export', 'TruAbutment Send', and '3D Model Export'. The main content area is titled 'Patient Data' and contains a table with the following data:

Case Name	Image Date	Last Updated	Array Size	Comments
[3-2-2021 11.19.34 AM]	3/2/21 6:52 PM	3/2/21 6:52 PM	-	-
Upper	3/2/21 6:52 PM	3/2/21 6:52 PM	-	Upper
Lower	3/2/21 6:52 PM	3/2/21 6:52 PM	-	Lower
Buccal1-7	3/2/21 6:52 PM	3/2/21 6:52 PM	-	Buccal1-7


Adding to an Existing Case

Select **Add to Open Case**.

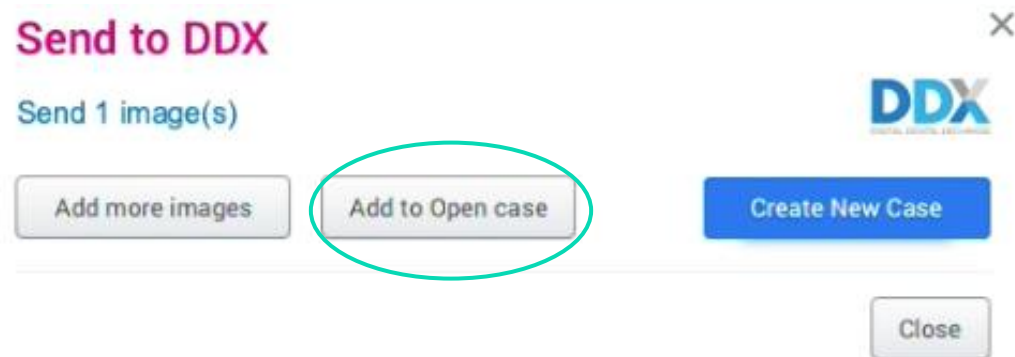

Adding to an Existing Case

The list will open showing all cases existing in DDX. Highlight the case and click **Send**.

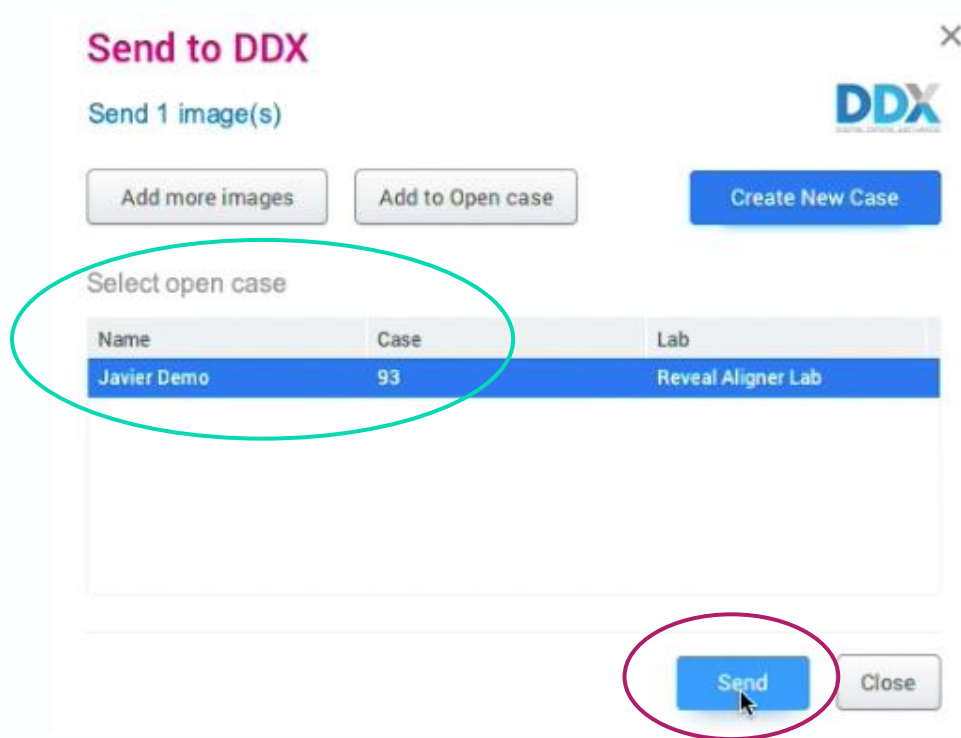

Send to DDX ×

Send 1 image(s) DDX

Add more images Add to Open case Create New Case

Select open case

Name	Case	Lab
Javier Demo	93	Reveal Aligner Lab

Send Close

Adding to an Existing Case

When the images have successfully uploaded to the DDX portal, you will receive a pop-up notification.



reveal[®]
CLEAR ALIGNERS

PLANMECA
Better care through innovation