



# Reveal trios digital integration guide



**Step 1.** Login to 3Shape  
Unite, click on the  
“More” tab and then “Store”

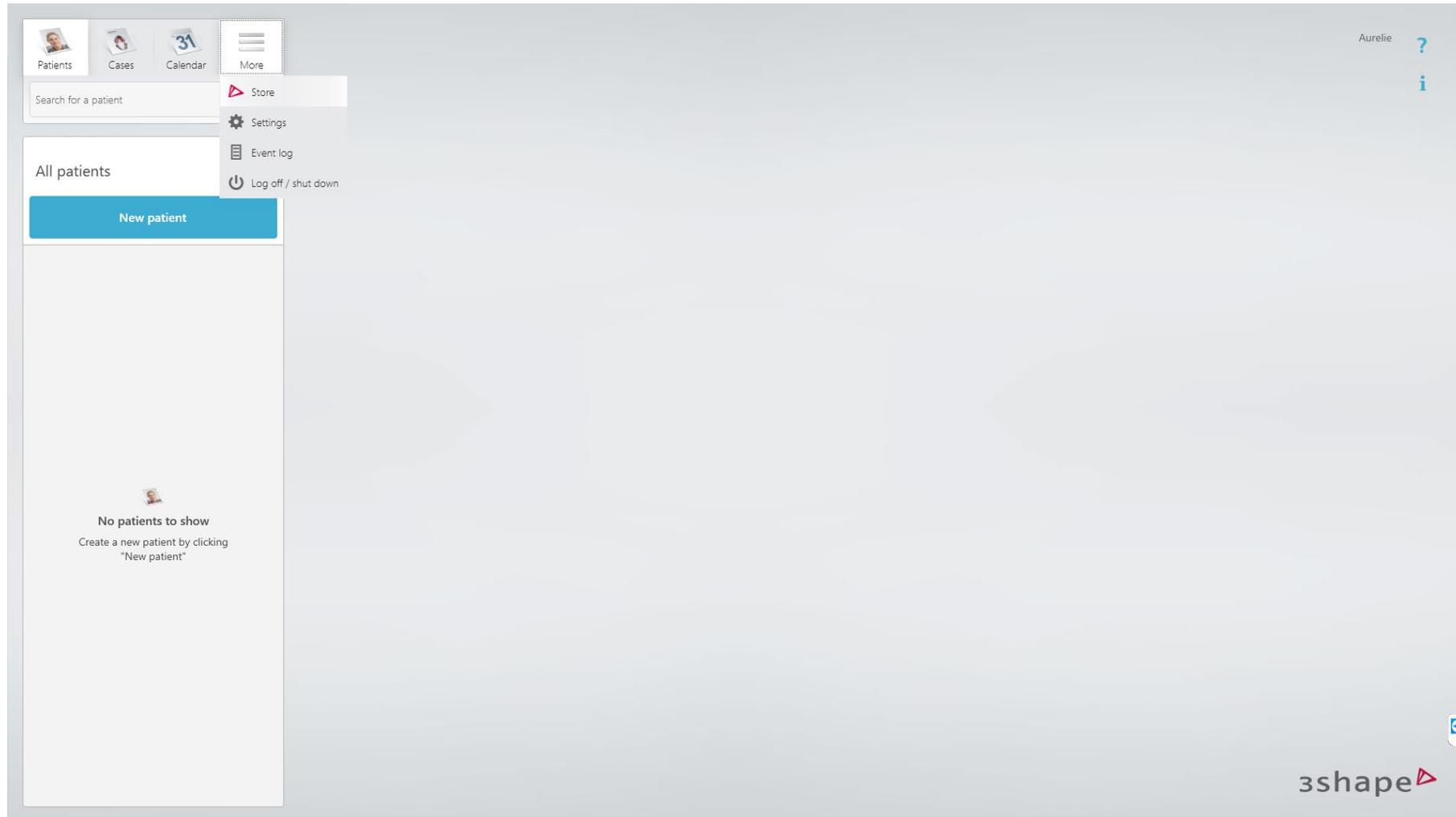


**Step 2.** In Store, Type Reveal in the search choose “Reveal Aligners HS Australia” and select “Install”.

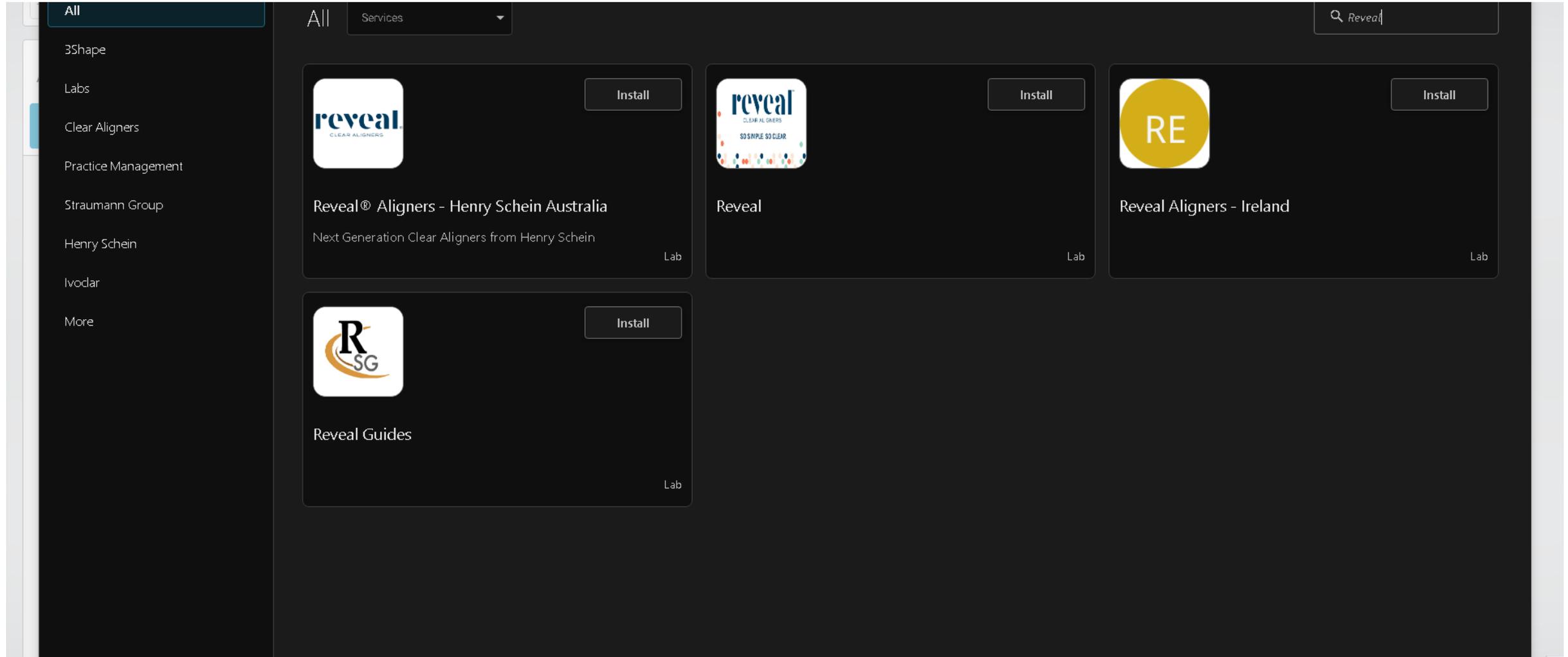


Step 3. Connect to lab : Click connect

The screenshot shows the Microsoft Store application window. On the left is a navigation pane with categories like 'All', '3Shape', 'Labs', 'Clear Aligners', 'Practice Management', 'Straumann Group', 'Henry Schein', 'Ivoclar', and 'More'. The main area displays a search for 'Reveal' with three results: 'Reveal® Aligners - Henry Schein Australia', 'Reveal', and 'Reveal Aligners - Ireland'. A 'Connect to lab' dialog box is open in the foreground, centered over the first result. The dialog has a title bar 'Connect to lab', a subtitle 'Message to Reveal® Aligners - Henry Schein Australia', and a text input field labeled 'Customize the message'. At the bottom of the dialog are 'Cancel' and 'Connect' buttons. The background shows the 'Reveal Guides' app card with a 'Lab' tag.

Step 4. Add Lab Connection: click ok

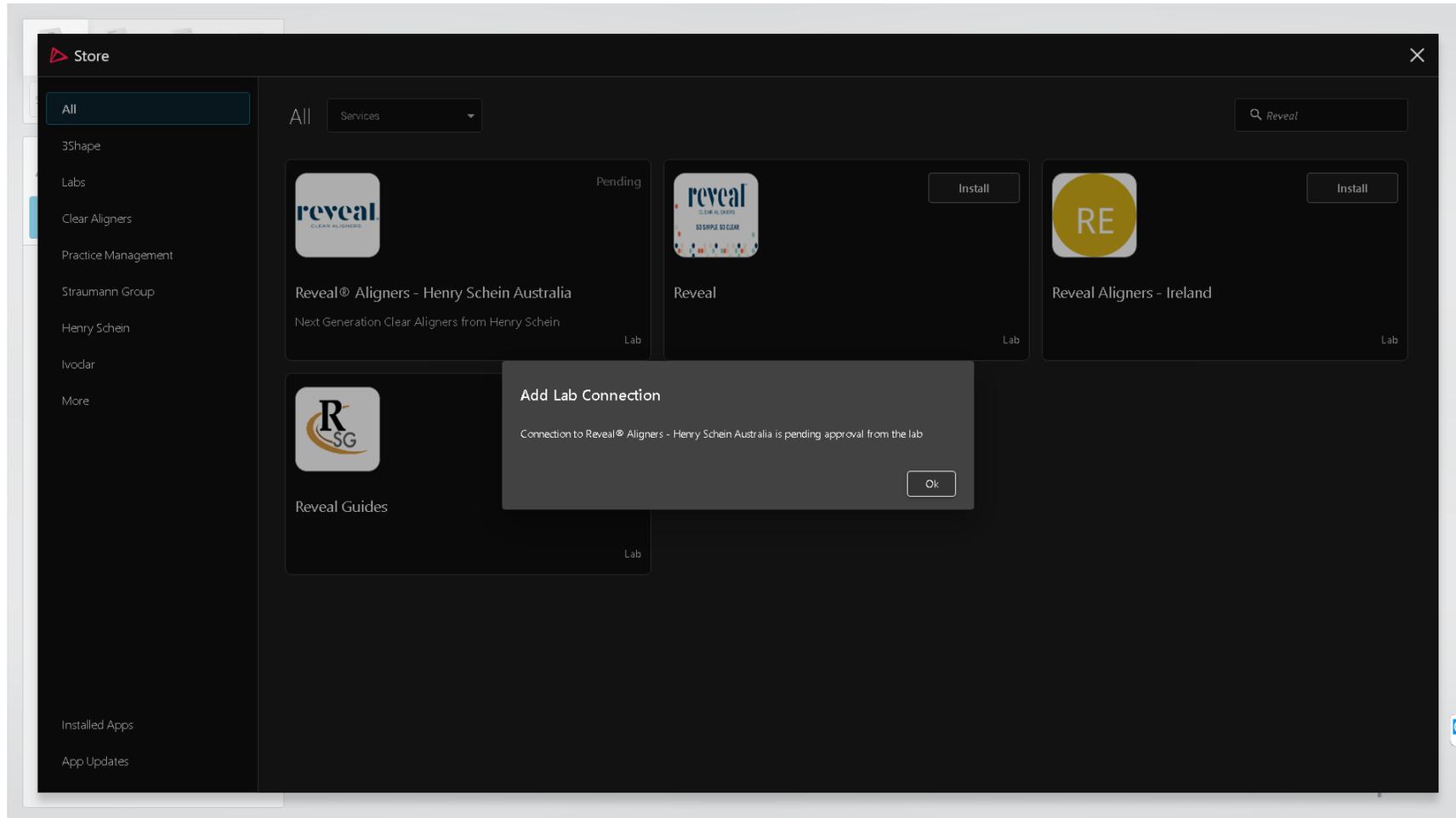


**Step 5.** The app connection will show as pending until the reveal lab approves your connection (within 1 business day)

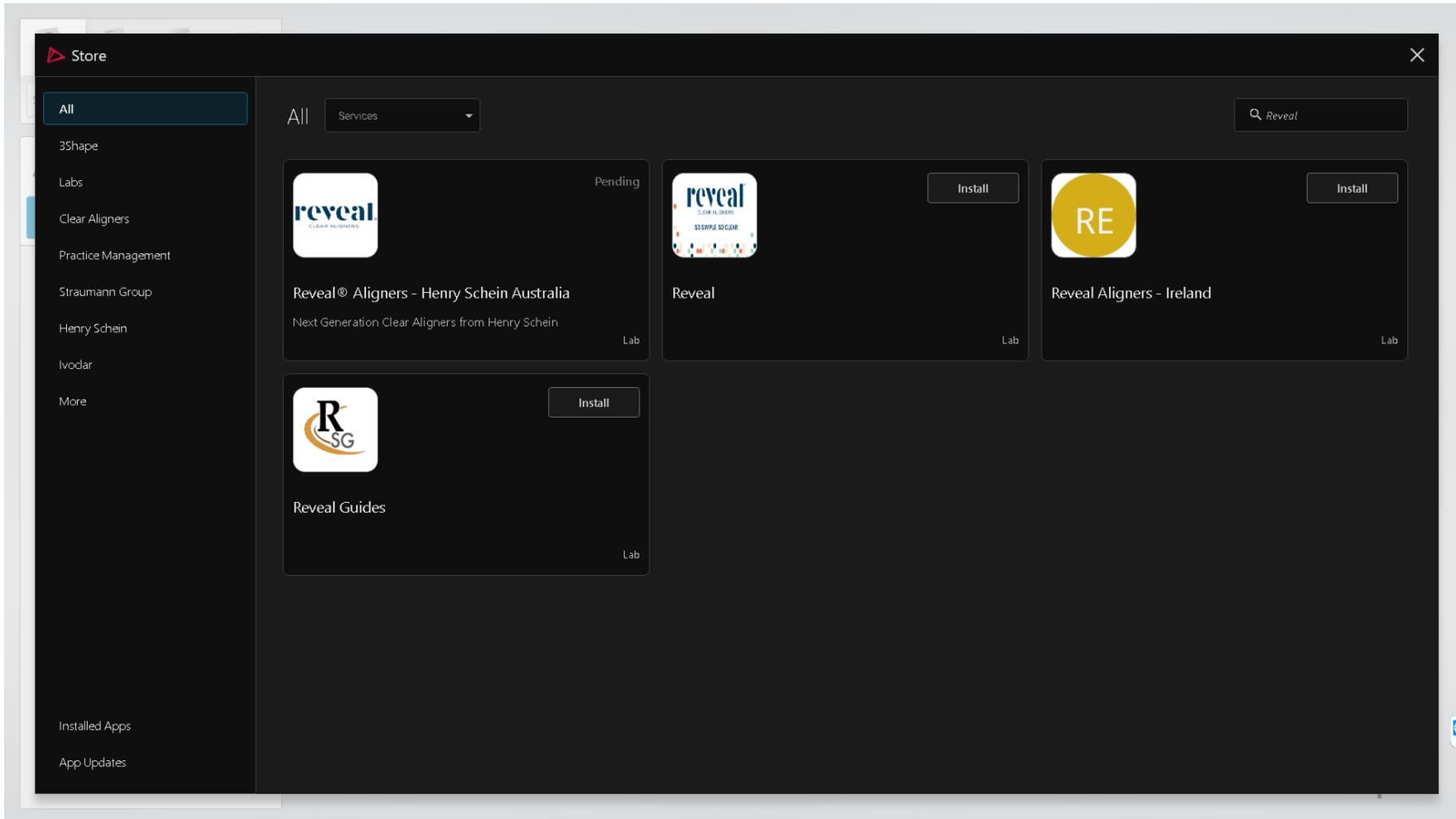

# Then go into your DDX account

**Step 4.** Login to DDX, click on the drop-down arrow on the top right hand side of the page, to choose Store.

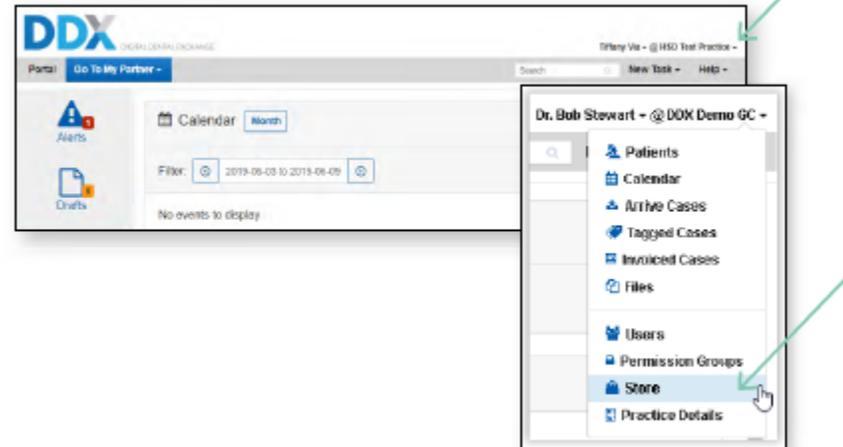

**Step 5. Scroll down and find TRIOS 3Shape Communicate and click on “Enable”.**



**Step 6. Click on “Configure your add-on”.**



**Step 7.** Click on “Authorize” and type in the email and password of your Communicate account.



The screenshot shows a dialog box titled "Configure 3Shape Communicate Integration". Below the title, there is a section labeled "3Shape Integration" which contains a text input field and a blue button with a plus sign and the text "Authorize with 3Shape".

**Step 8.** Now the two networks are connected! Before performing a TRIOS scan, select your Reveal Aligner Lab as the receiver and DDX will create a Draft case.



The screenshot shows the 3Shape Sign in page. At the top, the 3Shape logo is displayed. Below it, the text "Sign in" is centered. There are two input fields: the first is labeled "Email" with the placeholder "Original.com" and the second is labeled "Password" with the placeholder "123456789". Below the password field, there is a link that says "Forgot your password? (or activate account)". A red "Sign in" button is positioned below the link. At the bottom of the page, there is a "Forgot ID" link.

## Practice

- Patients
- Calendar
- Tagged Cases
- Invoiced Cases
- Files
- Arrive Cases
- Users
- Permission Groups
- Store
- Practice Details

## Configure 3Shape Communicate Integration

### 3Shape Integration

[- De-Authorize 3Shape](#)

**Enable Import**

**Patient Chart Match**



Tick enable import

[✓ Submit](#)



Click submit

### Communicate Case Browser

Case ID	Patient Name
Unable to get cases, try refreshing the page or re-authenticating with Threeshape	

[Previous Page](#)

Tips: tick “scan only” for the accurate scans to be uploaded onto DDX  
(not treatment simulation, not clear aligner)

